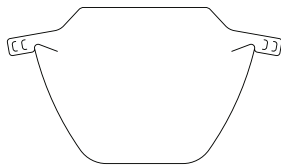
**PART A**  
PARTIE A

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
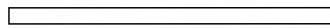
Shield / Bouclier

Head Strap / Sangle de tête

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
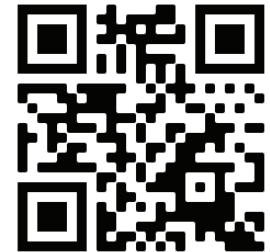
**Step 1: Inspect & Disinfect** - Use either a) alcohol based sanitizer/wipes, b) Benzalkonium Chloride or Hydrogen Peroxide containing cleaners\*, or c) 10:1 diluted sodium hypochlorite bleach solution (for 1 minute), wipe down both sides of all parts.

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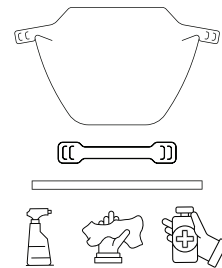
**Step 2: Assemble** - Wearing gloves, insert the left tab of the Shield into one of the slots on the left side of the Head Strap. Repeat this step with the right tab of the Shield and right side of the Head Strap.

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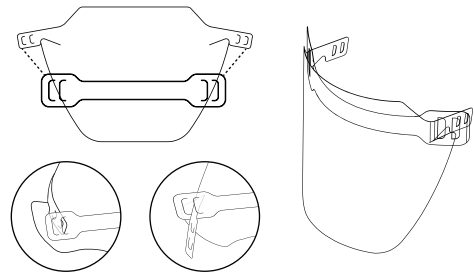**Step 3: Attach Elastic** - Locate the Elastic Band and weave it through the two slots on the Shield's tabs, from the inside slot to the outside. Repeat this step on the opposite side. Pull the ends of elastic band to tighten.

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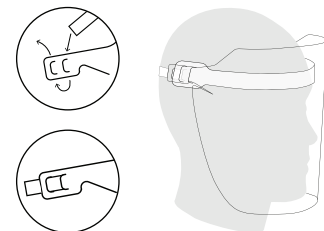
**PART B**  
PARTIE B

**SANITIZING INSTRUCTIONS**  
INSTRUCTIONS DE DÉSINFECTION


**Step 1: Disassemble** - Wearing gloves, disassemble your Shield, in the reverse order of the Assembly Instructions.

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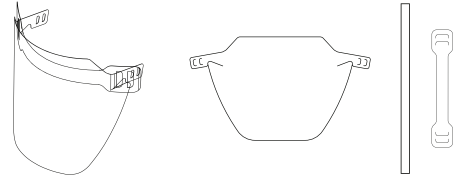**Step 2: Clean** - Using soap and warm water in a basin or sink, soak all parts of the Shield to remove any organic residue. (If the Shield will not be immediately reused, they can remain soaking indefinitely.)

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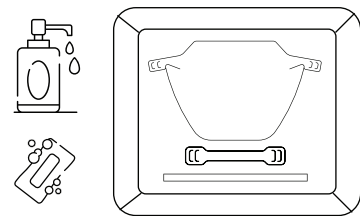
**Step 3: Disinfect** - Dry plastic parts with cloth or paper towel. Use either a) alcohol based sanitizer/wipes, b) Benzalkonium Chloride or Hydrogen Peroxide containing cleaners\*, or c) 10:1 diluted sodium hypochlorite bleach solution (for 1 minute), wipe down both sides of all parts.

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**Step 4: Dry & Store** - Store Shield in a sanitary environment, in a disassembled state until next use. Prior to next use, please refer to "Part A - Assembly Instructions" on reverse side.

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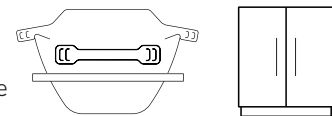Disclaimer: Personal protective equipment (PPE) is the last line of defense for protection against hazardous liquids, including airborne pathogens. It is not a substitute for more effective control methods and should be used only as a secondary measure of protection, not as the primary source. The Canadian Shield offers face and eye protection, however, it does not guarantee protection against infection, illness, disease or COVID-19. Therefore, there are no claims, representations or warranties, whether expressed or implied as to the safety of The Canadian Shield.

\* Please read and follow the instructions labelled on each cleaning product and always use approved disinfectants from: <https://www.canada.ca/en/health-canada/services/drugs-health-products/disinfectants/covid-19/list.html>

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