**1. Product Description**

**ACCESS II/ROTAMAX II**  
PN 2000003, 2000004

- 360 Swivel
- Nickel Plated Coating
- Single Port Spray
- Push Button
- Quick Connect

**ACCESS/ROTAMAX PRO KL**  
PN 2000021, 2000022

- Omni-directional lighting system
- Stainless steel body and head
- Quadruple spray
- Push button
- Easy grip coating

**ROTAMAX PRO 45KL/ROTAMAX PRO 45**  
PN 2000023, 2000024

- 20 Watts Power
- Ceramic German Bearings
- Rear Venting Back Cap
- Titanium Plated Finish
- 25,000 LUX Fused Optics (KL Only)

**2. Specifications**

Model	ROTAMAX II / ACCESS II	ROTAMAX PRO KL / ACCESS PRO KL	ROTAMAX PRO 45 KL / ROTAMAX PRO 45
Connection	2-Hole 360 Quick Connect (NO) or 4-Hole 360 Quick Connect (NO)	KaVo® MULTiflex® (FO)	KaVo® MULTiflex® (FO) / MW 4-Hole Fixed Back (NO)
Rotation Speed (rpm)	T>300,000; M>350,000	T>300,000; M>350,000	>350,000
Bur size (mm) (diameter / length)	Ø1.59-1.60 / T(19-25); M(16-21)	Ø1.59-1.60 / T(19-25); M(16-21)	Ø1.59-1.60 / 19-25
Driving Air Pressure (bar/psi)	2.6-3.0 / 37.7-43.5	2.6-3.0 / 37.7-43.5	2.6-3.0 / 37.7-43.5
Recommended Driving Air Pressure (bar/psi)	2.9 / 42	2.9 / 42	2.9 / 42
Spray Volume (ml/min)	>50	>50	>50
Water Pressure (bar/psi)	0.8-2.0 / 12.0-29.0	0.8-2.0 / 12.0-29.0	0.8-2.0 / 12.0-29.0
Chip Air Pressure (bar/psi)	1.0-3.0 / 14.5-43.5	1.0-3.0 / 14.5-3.5	N/A
Light system	N/A	Glass Rod	Glass Rod - KaVo® Type

\*T: torque head; M: mini head; FO: fiber optic; NO: non-optic

\*\*Chip air pressure must be greater than chip water pressure and both values need to be set at the same time

**3. Safety Notes**

1. ⚠ Once handpiece is damaged (ie deformed caused by hit) or improperly used (ie. sterilized above suggested temperature), be sure not to continue using it.
2. Before operation, always make sure the product and its accessories meet the specified requirements (ie drive air pressure, lubrication...etc).
3. Please contact our dealer or us with proof of purchase, whenever you have problems such as abnormal vibration, exceeding temperature or unusual noise...etc.
4. Avoid contact between the instrument head and soft tissue (risk of burning due to the press button heating up).

**4. Installation**

Fig. 1 Install EZclick connector

Fig. 2 Rotate the connector to install

Fig. 3 Strip EZclick connector

To install or remove handpiece, please follow the steps illustrated from Fig. 1 to Fig. 3 (Please refer to the coupling manufacturers' instructions.)

**5. Changing Bur**

1. Follow the direction of the arrows shown in attached image (Fig. 4) to install or remove bur.
2. Put on protective gloves to reduce the risk of interactive infection, especially not to touch the used burs without proper protection.
3. Make sure bur is tightly secured before operation, otherwise risks of injury to doctors, patients or the third party could occur.
4. Bur must be complied with ISO requirement. Burs not in good conditions such as being bent and damaged, or are deviated from ISO standards and in improper length may decrease the lifetime of the handpiece or even cause severe consequence such as injury to people.
5. Do not press the button while bur is rotating, otherwise it could harm internal parts.

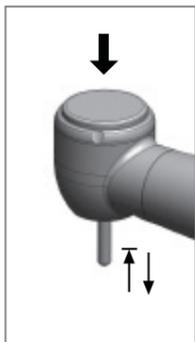
Fig. 4 Changing Bur

**6. Replacement of Turbine**

1. Please contact Sable Industries (or dealer) and let technicians replace the Turbine.
2. Do not use unsuitable Turbines or spare parts, otherwise could damage the handpiece and result in severe consequence during treatment.

**7. Cleaning**

\*Clean and lubricate the handpiece as soon as possible after treatment. Failure to properly maintain the handpiece may cause infection, overheating leading to burn injuries. The use of automated cleaning devices is not recommended. ⚠ DO NOT IMMERSIVE INSTRUMENTS IN WATER, DISINFECTANT SOLUTION OR CLEAN IN ULTRASONIC DEVICES

1. Protective gloves should be worn every time hand piece is to be cleaned and lubricated.
2. Detach the handpiece from the coupler and remove the bur.
3. Use clean cloth with mild cleaning solution to clean the handpiece surface carefully to remove any residue.
4. Operate the handpiece at the chair-side for at least 20 seconds to purge fluids.  
\*Wash the handpiece and irrigation nozzle under running water following the procedures below. (Water condition: <38°C (100°F), >3.5L/min; water should be same quality as drinking water)
5. Clean the external surfaces of the handpiece with a soft bristled toothbrush for 15 seconds or more.
6. Clean around the bur insertion hole for 15 seconds or more, using an interdental brush (wire thickness 0.7mm) that conforms to size 4 in ISO 16409.
7. Under appropriate lighting (natural office lighting), inspect the handpiece for blood or other debris. If any visible debris remain, repeat the process until the handpiece is visually clean.
8. Wipe the exterior of the handpiece clean using a cloth moistened with disinfectant or ethanol with a concentration of 60-90% (ie. Isopropyl)

Fig. 5 Cleaning

**8. Lubrication and Maintenance**

1. ⚠ Use dental handpiece lubrication oil for maintenance, any unspecified oil not intended for lubricating handpieces could shorten the lifetime and harm human body.
2. To lubricate High Speed Series handpiece, put Spray Nozzle into nozzle of the lubrication oil can (Fig. 6). Connect handpiece with the nozzle (Fig. 7) and inject lubrication oil for ~2-3 seconds.
3. Be sure to lubricate the handpieces prior to sterilization and run the handpieces for ~20-30 seconds to emit residual oil and dirt.
4. To lubricate handpiece chucks, put Shaft Spray Nozzle into nozzle of the lubrication oil can and inject lubrication oil into the chuck (Fig. 8) for 1 second. It's recommended to lubricate chuck at least once a week and then proceed step 2.
5. After unpacking and prior to first use, be sure to lubricate with oil and run the handpieces in low rotation speed for 30 seconds and in full speed for ~20-30 seconds to emit residual oil.

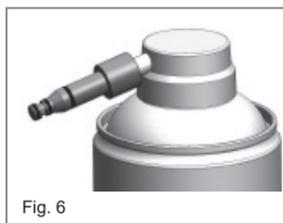
Fig. 6

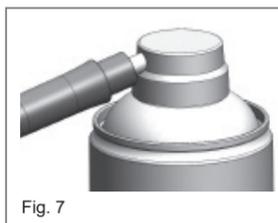
Fig. 7

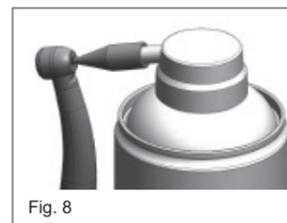
Fig. 8

**9. Sterilization**

Sable suggests sterilization in accordance with ISO 117373-2-2019

1. ⚠ Place handpiece in single-use sterile bag and proceed steam prevacuum sterilization. Do not open the sterile bag before use in order to avoid contact infection.
2. ⚠ Use qualified sterilization equipment specified by the authority/government for sterilization procedures.
3. ⚠ Do not use chemical steam nor immerse handpiece or cartridge/rotor in disinfectant solution, otherwise could damage the handpiece and cause severe consequences.
4. ⚠ \* For 2000021, 2000022, 2000023, 2000024 Sterilization PreVac cycle is a minimum of 4 minutes at 134°C. (drying time is 15 minutes).  
⚠ \* For 2000003, 2000004 Sterilization PreVac cycle is a minimum of 8 minutes at 135°C. (drying time is 40 minutes).

**10. Disclaimer**

1. Information in this operation manual were carefully reviewed. We cannot assume any liability for any damage arising from improper use of the handpiece or from use of this manual.
2. Products are subject to change without notice.
3. Indications for Use: High Speed Series Handpieces are intended for removing carious parts of tooth, proceeding cavity and crown preparations.

KaVo® & MULTiflex® are the registered trademarks of Kaltenbach & Voigt GmbH CO., Germany.

CAUTION: U.S. Federal law restricts the use of this device to licensed professionals.(Rx Only)

**11. Quality and Service**

All of our products have passed strict QC procedures before delivery. Please note that improper use or not using the recommended parts and accessories or modifying our products without our approval may shorten the lifetime of the product or cause damage. Please contact us or dealers with proof of purchase for consultation whenever you have any questions.

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