SPECIAL PRECAUTIONS

Caution: Sterilize before first time use and every subsequent use thereafter (see sterilization instructions).

Caution: U.S. Federal law restricts this device to sale by or on the order of a dentist.

1. Ensure that the frequency (25K or 30K) of the scaler unit is suitable for this insert.
2. Always use a light, smooth, gentle touch with a well-positioned stable intraoral or extraoral fulcrum. The lateral surfaces of the insert, is normally used for clinical procedures.
3. In the event that this insert is dropped or damaged or becomes bent, discard the insert and replace with a new insert. Inserts with high usage, which exceed the warranty period, should be discarded to avoid in-use breakage and possible injury.
4. These inserts should not be used for other procedures including condensation of amalgam.
5. Care must be exercised to avoid iatrogenic results from the use of ultrasonics in the presence of dental restorations.
Note that the tip end or other sharp aspect of an insert can scratch or otherwise modify the surface of gold or amalgam restorations. Thin porcelain veneers and composite margins can be cracked or crazed by activated ultrasonic tips.
6. Normal wear of the tip occurs during function. When approximately 2 mm of the tip has worn the insert may lose up to 50% efficiency, and should be discarded. Contact with composite restorations will cause more rapid attrition of ultrasonic inserts.

DEVICE DESCRIPTION

Ultrasonic inserts are accessory inserts designed for use with magnetostrictive scaler units, (see specifications) that generate a frequency of either 25KHz or 30KHz.

A range of tip designs are available to suit different clinical conditions. Finger – grips are either plastic or metal and the water channel is either of internal or external design.

These inserts are designed to be used in magnetostrictive ultrasonic scalers only by professionally trained operators such as Dentists or Dental Hygienists to perform general supra and subgingival scaling procedures.

SPECIFICATIONS

These inserts are designed for 25k or 30k magnetostrictive scaler handpieces with:- Internal diameter of 7.3 mm. - Internal length of no less than 125 mm.

Internal water channel with plastic finger-grip

DF#3 Flat Heavy to moderate supragingival deposits on buccal and lingual surfaces.

DF#10 Universal Heavy to moderate supragingival deposits, light to moderate subgingival deposit on all surfaces.

DF#1000 3 Bend Light to moderate supragingival deposits especially interproximally.

External water channel with metal finger-grip

Tip Design Application

DP#10 Universal Heavy to moderate supragingival deposits and light to moderate subgingival deposit on all surfaces.

DP#100 Universal Light subgingival deposits on all surfaces.

WATER FLOW

An adequate water flow setting, indicated by a small spray or rapid drips, should be selected for the power setting used, so as to prevent overheating of the insert tip during function, and to prevent possible injury.

O-RINGS

Before placing inserts into the handpiece performing the following steps will increase 'O' ring life.

- 1 - Fill entire handpiece with water
- 2 - Lubricate o-ring (black or green material) with water
- 3 - Gently twist the insert down into the handpiece until fully seated.

UNIT POWER SETTINGS

DP#100, & DF#100 inserts should be used with minimum power setting only on the unit

DF#10, DF#1000& DP#10 inserts should be used on low - medium power setting. DF#3 inserts may be used on medium - high power setting

As a rule the power setting on the unit should always be set at the minimum and increased only as dictated by clinical requirements.

STERILIZATION

The handpiece does not come sterilized from the manufacturer. it is also recommended That the handpiece or motor be sterilized after each use. FDA/Health Canada autoclaving or sterilization packaging is recommended to preserve item in sterile condition.

1. Wipe off debris on the entire tool surface with alcohol-soaked cloth. Phenolic Alcohol, up to 3% glutaraldehyde based on the agents containing acetone or chlorine.
2. Lubricate (See Lubrication)
3. Follow the instructions for your particular Autoclave or Sterilizer

CYCLES:

The Handpiece scaler tip can be subject to 250 cycles = (Performed Sterilization Process's) per EN ISO 14457 requirement 5.10, before showing signs of deterioration. When there is a noticeable loss in the power or performance of the Handpiece scaler tip, it should be immediately returned to your authorized dealer or repair service.

LIMITED WARRANTY

This ultrasonic insert has been manufactured to meet stringent quality assurance requirements. If, in normal use, any insert is found to be defective in material or workmanship within a period of ninety (90) days from the date of original purchase, it will be replaced.

The limited warranty stated herein is the sole and exclusive warranty with respect to the insert, and hereby disclaims any and all other warranties, expressed or implied, including, without limitation, any statutory warranties or any warranties of merchantability, fitness for use or fitness for any particular purpose.

Manufacturer or supplier will not be responsible for any inconvenience loss, injury, or direct, indirect, tort, exemplary, punitive, special incidental or consequential damage arising from the possession or use of the insert. In no event will liability exceed the purchase price of the insert.

Tampering, abuse, misuse, neglect, alteration, accidental damage, failure to follow manufacturer's instructions, improper disinfection or sterilization procedures, bending, re-grinding or re-shaping, or lack of reasonable care with respect to the insert will void this warranty. Use of the insert by anyone other than a licensed dental practitioner or qualified hygienist will void this warranty.

Manufacturer or supplier will not be responsible for damage in transit, and any claim for such damage should be made promptly against the carrier. Shipping charges are to be paid by the sender (shipper). The return must be accompanied by a written description of the claimed defect. In addition, a warranty claim must be accompanied by a dated sales receipt or other appropriate documentation showing the date of purchase. In the event of a warranty claim, it is the responsibility of the user to return the defective insert(s) to an approved agent.

Sable Industries Inc.

100 Campbell Ave Unit 5 Kitchener ON N2H 4X8, Canada

Phone 1-800-368-8106 or see Sables Website

for other products and product information.

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