SABLE INDUSTRIES INC.✱

SABLE SLOW SPEED CONTRA ANGLES & NOSECONES MAINTENANCE & CLEANING MANUAL

1600202, 1600203, 1600204, 1600210, 1600211, 1600212, 1600214, 1600215, 1600216,
1600217, 1600218, 1600302, 1600303, 1600304, 1600310, 1610210, 1610310

SPECIAL PRECAUTIONS:

1. Always exercise extreme caution when operating your Attachment and Head
2. Always inspect your Attachment and Head thoroughly before each use.
3. Always use PPE (Proper Protective Equipment) when operating and cleaning your Sable Slowspeed.
4. See your Occupational Health and Safety Act or other safety ordinances for specific instructions.
5. This is a precision instrument. Do not drop or apply excessive force when handling.
6. If product is dropped your dealer should be contacted, it is recommended that you do not use.
7. Contact your dealer right away if any problem should occur with any of your Sable products.
8. Sterilize before first time use and before every subsequent use thereafter.
9. This product is for use by a Dentist or trained professional in the dental field.

HOW TO USE:

1. Attachment of E-Type Contra Angles or Nosecones. Slide Attachment onto shaft of Air Motor until locked in place.
2. To remove the Contra Angle or Nosecone, pull attachment, or push the release button located on the exterior of Air Motor.

CYCLES:

The Contra Angle and Nosecone can be subject to 250 cycles = (Performed Sterilization Process) per EN ISO 14457 requirement 5.10 before showing signs of deterioration.

FEATURES & SPECIFICATIONS:

1. Contra Angles and Nosecones conform to ISO 14457 specifications.
2. Connection with Attachments: meets ISO-9168 standard for E-Type Connections
3. Method of Sterilization: Autoclave

CLEANING: Preparation and Cleaning before Sterilization:

**Clean and lubricate the handpiece as soon as possible after treatment. Failure to properly maintain the handpiece may cause infection, overheating leading to burn injuries.  DO NOT IMMERSIVE INSTRUMENTS IN DISINFECTANT SOLUTION OR CLEAN IN ULTRASONIC DEVICES*

1. PREPARATION:

- a) Always wear protective gloves, a mask, and protective goggles for safety purposes and to minimize the risk of infection.
- b) Remove the bur. (if applicable)
- c) Wipe the exterior of the handpiece using a cleaning wipe.
- d) Operate the handpiece at the chair-side for at least 20 seconds to purge fluids in the handpiece.
- e) Remove the handpiece, carry the handpiece to the decontamination area and proceed to step 2.

2. CLEANING AND DRYING:

**Wash the handpiece and irrigation nozzle under running water following the procedures below. (Water condition: <38°C (100°F), >3.5L/min; water should be same quality as drinking water)*

- a) Clean the external surfaces of the handpiece/accessory with a soft bristled toothbrush for 15 seconds or more.
- b) Clean around the bur insertion hole for 15 seconds or more, using an interdental brush (wire thickness 0.7mm) that conforms to size 4 in ISO 16409. (if applicable)
- c) Under appropriate lighting (natural office lighting), inspect the handpiece/accessory for blood or other debris. If any visible debris remain, repeat the process until it is visually clean.
- d) Wipe the exterior of the handpiece/accessory clean using a cloth moistened with disinfectant or ethanol with a concentration of 60-90% (ie. Isopropyl)

LUBRICATION:

USE ONLY AUTOCLAVABLE HANDPIECE LUBRICANT.

1. Remove contra angle or Nosecone from motor (leave motor connected to the airline).
2. Using spray lubrication can or lubrication station, apply lubricant into contra angle or nose cone.
3. Reattach entire contra angle or nosecone to air motor if not using a lubrication station.
4. Run entire tool at normal speed for 10 seconds.
5. Wipe off excess oil from entire tool surface.

AUTOCLAVING PROCEDURE:

Sterilize in accordance with ISO17665-1:2006. Suitable autoclaving packaging is recommended to preserve item in sterile condition. Once the product has been sterilized it should be used as at earliest convenience to ensure sterilization is maintained.

1. Clean before sterilization (See Cleaning).
2. Lubricate before sterilization (see LUBRICATION above).
3. For part numbers, **1600210, 1600211, 1600212, 1600214, 1600215, 1600216, 1600217, 1600218, 1600310, 1610210, 1610310** set parameters for Moist Heat Sterilizer. PreVac cycle 134 degrees Celsius for 8 minutes and 20 minute dry time .
4. For part numbers, **1600202, 1600203, 1600204, 1600302, 1600303, 1600304** sterilization parameters for moist heat sterilizer pre vac cycle to 132 degrees Celsius for 10 minutes dry time 40 minutes.
5. Use Sterilized head as soon as possible to ensure sterilization is maintained.

WARRANTY INFORMATION:

Sable Slow Speed Contra Angles and Nosecones are warranted to be free of manufacturing defects for a period of 6 months* from date of purchase for the standard Contra Angles and Nosecones and 12 months for the Premium Contra Angles and Nosecones

**Warranty is conditional on proper use and maintenance of the Contra Angles and Nosecones. Warranty is limited to repair or replacement at manufacturers' discretion. Under no circumstances will the manufacturer be held responsible for incremental or consequential damages. Please contact point of purchase authorized distributor for warranty repair.*

Disclaimer: Sable Industries Inc. recommends that the sterilization process outlined in the Directions for use (DFU) described in this document be followed precisely. Sable Industries Inc. is not responsible for any products supplied in which the end user does not follow the DFU.

Sable Industries Inc.

100 Campbell Ave. Suite 5 Kitchener, ON N2H 4X8, Canada
See Sables Website for other Sable products and product information
www.sableindustriesinc.com or Phone 1-800-368-8106

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