SABLE

INDUSTRIES INC.

Manufacturing Information

Your new turbine has been manufactured to exacting specifications to meet and exceed the International Handpiece Standards ISO 14457.

Indication for use

This product is part of a dental HP used by a dental professional for the purpose of preparation of dental cavities for restoration.

Warranty Install Information

Sable brand turbines are warranted for a period of 12 months (12 months on bearings) from the date of install, the warranty covers gripping force, rotor eccentricity, balance and bearing performance.

Important: We are able to offer this warranty on our bearings partially due to new autoclavable grease installed at the factory.

To ensure distribution of the autoclavable grease within the bearing and to ensure extended bearing life; after install, operate the hand-piece at ½ normal running pressure for 1 minute, then at full pressure for 1 minute.

Care and Maintenance:

The Turbine does not come sterilized from the manufacturer. It is also recommended that the Turbine be sterilized after each use.

FDA/Health Canada autoclaving or sterilization packaging is recommended to preserve item in sterile condition.

1. Wipe the exterior of the turbine clean using a cloth moistened with disinfectant or ethanol with a concentration of 60-90% (ie. Isopropyl)
2. Lubricate
3. Minimum cycle time = 132°C for 4 minutes. Drying time of a minimum 20 minutes

**(The exception to the entire above rule is
“all turbines with lube free Bearings Do not require lubrication”)**

100 Campbell Avenue, Suite #5, Kitchener, Ontario N2H 4X8, Canada

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