

## Liquid Bio-Pure Mixing Instructions, Per Operator

Dental Ops	Between Patient Flush <i>*As needed bewtween patients*</i>		Daily Maintenance (Liquid) <i>*These instructions based on a 4-day work week*</i>		Twice Weekly Maintenance (Liquid)	
	Solution (oz./mL)	Water / oz. & mL	Solution (oz./mL)	Water / oz. & mL	Solution (oz./mL)	Water / oz. & mL
1	1/4oz. (7.5mL)	24oz. / 710mL	1/4oz. (7.5mL)	24oz. / 710mL	1/2oz. (15mL)	48oz. / 1.419L
2	1/2oz. (15mL)	48oz. / 1.419L	1/2oz. (15mL)	48oz. / 1.419L	1oz. (30mL)	96oz. / 2.839L
3	3/4oz. (22.5mL)	72oz. / 2.129L	3/4oz. (22.5mL)	72oz. / 2.129L	1.5oz. (45mL)	144oz. / 4.257L
4	1oz. (30mL)	96oz. / 2.839L	1oz. (30mL)	96oz. / 2.839L	2oz. (60mL)	192oz. / 5.678L
5	1 1/4oz. (37.5mL)	120oz. / 3.549L	1 1/4oz. (37.5mL)	120oz. / 3.549L	2.5oz. (75mL)	240oz. / 7.095L
6	1 1/2oz. (45mL)	144oz. / 4.259L	1 1/2oz. (45mL)	144oz. / 4.259L	3oz. (90mL)	288oz. / 8.517L
7	1 3/4oz. (52.5mL)	168oz. / 4.968L	1 3/4oz. (52.5mL)	168oz. / 4.968L	3.5oz. (105mL)	336oz. / 9.937L
8	2oz. (60mL)	192oz. / 5.678L	2oz. (60mL)	192oz. / 5.678L	4oz. (120mL)	384oz. / 11.356L
9	2 1/4oz. (67.5mL)	216oz. / 6.388L	2 1/4oz. (67.5mL)	216oz. / 6.388L	4.5oz. (135mL)	432oz. / 12.776L
10	2 1/2oz. (75mL)	240oz. / 7.098L	2 1/2oz. (75mL)	240oz. / 7.098L	5oz. (150mL)	480oz. / 14.195L
11	2 3/4oz. (82.5mL)	264oz. / 7.807L	2 3/4oz. (82.5mL)	264oz. / 7.807L	5.5oz. (165mL)	528oz. / 15.618L
12	3oz. (90mL)	288oz. / 8.517L	3oz. (90mL)	288oz. / 8.517L	6oz. (180mL)	576oz. / 17.037L
13	3 1/4oz. (97.5mL)	312oz. / 9.227L	3 1/4oz. (97.5mL)	312oz. / 9.227L	6.5oz. (195mL)	624oz. / 18.456L
14	3 1/2oz. (105mL)	336oz. / 9.937L	3 1/2oz. (105mL)	336oz. / 9.937L	7oz. (210mL)	672oz. / 19.875L
15	3 3/4oz. (112.5mL)	360oz. / 10.647L	3 3/4oz. (112.5mL)	360oz. / 10.647L	7.5oz. (225mL)	720oz. / 21.294L
16	4oz. (120mL)	384oz. / 11.356L	4oz. (120mL)	384oz. / 11.356L	8oz. (240mL)	768oz. / 22.712L
17	4 1/4oz. (127.5mL)	408oz. / 12.066L	4 1/4oz. (127.5mL)	408oz. / 12.066L	8.5oz. (255mL)	816oz. / 24.132L
18	4 1/2oz. (135mL)	432oz. / 12.776L	4 1/2oz. (135mL)	432oz. / 12.776L	9oz. (270mL)	864oz. / 25.552L
19	4 3/4oz. (142.5mL)	456oz. / 13.486L	4 3/4oz. (142.5mL)	456oz. / 13.486L	9.5oz. (285mL)	912oz. / 26.972L
20	5oz. (150mL)	480oz. / 14.195L	5oz. (150mL)	480oz. / 14.195L	10oz. (300mL)	960oz. / 28.390L

**\*Bio-Pure should be mixed with warm water and used at the end of the work day\***